


REF. 430CH

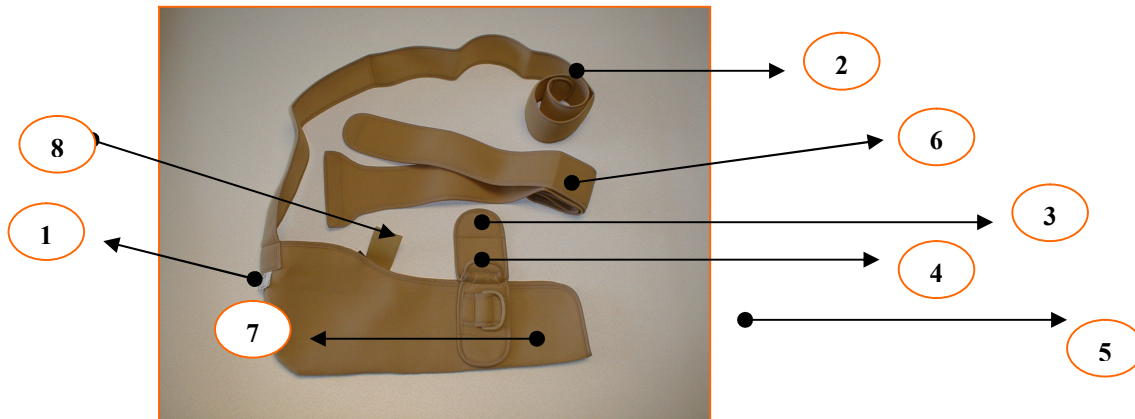

- | | | | |
|-----------------|---------------------------|------------------|--------------------|
| 1.- Arm support | 2.- Suspender | 3.- Ring part | 4.- Ring A |
| 5.- Ring B | 6.- Immobilizer suspender | 7.- Closing area | 8.- Velcro closing |



- 1.- Slide the arm into the support (1).
- 2.- Close the Velcro (8)
- 3.- Position the ring parts



- 4.- Make the suspender (2) pass over the shoulder.
- 5.- Make the suspender (2) pass through the Ring A (4) and close it. The arm must remain at a 90-degree angle



- 6.- Close the immobilizer suspender (6) on the widest area of the elbow (7).



- 7.- Wrap the back immobilizer suspender around the body and make it pass through the Ring B (5). Adjust and close.